


AED Plus® Label Application Instructions

2010 American Heart Association Guidelines Update

Label set contains two labels that replace the "Open Airway" and "Check Breathing, Give two Breaths" Icons.

Follow the instructions below for proper application.

1. Remove labels from outer packaging.
(Both labels are on one sticker)
 - a. Overlay label for "Open Airway" Icon
 - b. Overlay label for "Check Breathing, Give two Breaths" Icon

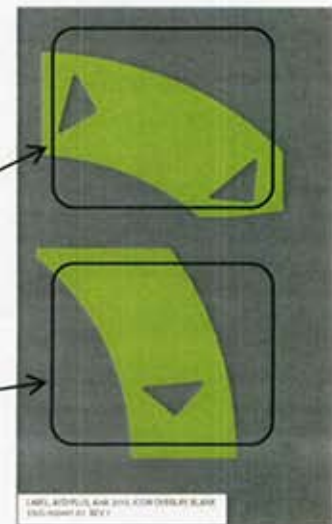

2. Remove overlay label from backing for "Open Airway" and carefully place over icon:



3. Remove overlay label for "Check Breathing, Give two Breaths" and carefully place over icon:



Once labels are applied, the AED Plus faceplate should look similar to this.